

**BIOLOGICAL HEALTH RISKS  
QUALITY OF LABORATORIES**

**CLINICAL BIOLOGY  
COMMITTEE OF EXPERTS**

**EXTERNAL QUALITY ASSESSMENT  
IN CLINICAL BIOLOGY**

**DEFINITIVE GLOBAL REPORT**

**Molecular Microbiology**

**HBV-HCV**

**SURVEY 2023/S5**

**Sciensano/Molecular Microbiology (HBV-HCV)-2-E**

Biological health risks  
Quality of laboratories  
J. Wytsmanstreet, 14  
1050 Brussels | Belgium

[www.sciensano.be](http://www.sciensano.be)

<b>COMMITTEE OF EXPERTS</b>
-----------------------------

<b>Sciensano</b>					
Secretariat		PHONE:	02/642.55.22	FAX:	02/642.56.45
		e-mail:	ql_secretariat@sciensano.be		
Bernard China	Scheme coordinator	PHONE:			
		e-mail:	Bernard.china@sciensano.be		
Kris Vernelen	Alternate coordinator	PHONE:			
		e-mail:	Kris.vernelen@sciensano.be		
<b>Experts</b>	<b>Institute</b>				
Marijke Reynders	<b>AZ sint Jan</b>				
Stefanie Desmet	<b>ULeuven</b>				
Veerle Matheeussen	<b>UZA</b>				
Samy Mzougui	<b>CHU Liège</b>				
Walter Verstrepen	<b>ZNA</b>				

A draft version of this report was submitted to the *experts and to the NRC* on: 30/11/2023.

**Authorization of the report:** by Bernard China, scheme coordinator

**Date of publication: 15/12/2023**

All the reports are also available on our webpage:

- NL: <https://www.sciensano.be/nl/kwaliteit-van-laboratoria>
- FR: <https://www.sciensano.be/fr/qualite-des-laboratoires>
- EN: <https://www.sciensano.be/en/quality-laboratories>

# TABLE OF CONTENTS

## TABLE OF CONTENTS

<b>1</b>	<b>INTRODUCTION</b> .....	<b>4</b>
1.1	<b>The samples</b> .....	<b>4</b>
1.1.1	Production .....	4
1.1.2	Homogeneity .....	4
1.1.3	Stability .....	4
1.2	<b>Scoring system</b> .....	<b>5</b>
1.2.1	Qualitative results .....	5
1.2.2	Quantitative results .....	5
1.3	<b>Survey dates</b> .....	<b>5</b>
<b>2</b>	<b>RESULTS</b> .....	<b>6</b>
2.1	<b>HBV</b> .....	<b>6</b>
2.1.1	the participants .....	6
2.1.2	Results per sample .....	6
2.1.3	Results per method .....	7
2.2	<b>HCV</b> .....	<b>8</b>
2.2.1	The participants .....	8
2.2.2	Qualitative results .....	8
2.2.3	Quantitative results .....	8
2.2.4	Genotyping .....	9
<b>3</b>	<b>ANNEXES</b> .....	<b>10</b>

# 1 INTRODUCTION

## 1.1 The samples

The samples were prepared from a negative serum (HBV and HCV negative) spiked or not with HBV or HCV positive sera. These positive patient sera were provided by the NRC (UCL Saint-Luc, Brussels).

### 1.1.1 PRODUCTION

100 µl of sample of patient 1 (sample 19292) (HBV) was diluted in 110 mL of negative serum (13155) and then divided in 2 mL aliquots to constitute sample HBV23-1.

100 µl of sample of patient 2 (sample 19289) was diluted in 110 mL of negative serum (13155) and then divided in 2 mL aliquots to constitute sample HBV23-2.

The negative serum (13155) was divided in 2 mL aliquots to constitute sample HBV23-3.

200 µl of serum of patient 3 (sample 19294) (HCV) was diluted in 110 mL of negative serum (13155) and then divided in 2 mL aliquots to constitute sample HCV23-1.

100 µl of serum of patient 4 (sample 19296) (HCV) was diluted in 110 mL of negative serum (13155) and then divided in 2 mL aliquots to constitute sample HCV23-2.

The negative serum (13155) was divided in 2 mL aliquots to constitute sample HCV23-3.

Therefore the panel HBV consists of 3 samples (HBV23-1, HBV23-2, HBV23-3) and the panel HCV consists of 3 samples (HCV23-1, HCV23-2, HCV23-3).

### 1.1.2 HOMOGENEITY

Randomly selected samples were sent to the NRC to check the homogeneity.

The sample HBV23-1 was detected as positive (6.20 Log IU/mL)

The sample HBV23-2 was detected as positive (5.31 Log IU/mL)

The sample HBV23-3 was detected as negative.

The sample HCV23-1 was detected as positive (4.06 Log IU/mL)

The sample HCV23-2 was detected as positive (3.69 Log IU/mL)

The sample HCV23-3 was detected as negative.

The samples were considered as homogeneous.

### 1.1.3 STABILITY

The results before and during the surveys were compared.

Table I.1. Samples stability.

Sample	Result pre-PT	Result PT
HBV23-1	Positive (6.2 log IU/mL)	Positive (6.55 Log IU/mL)
HBV23-2	Positive (5.31 IU/mL)	Positive (5.52 IU/mL)
HBV23-3	Negative	Negative
HCV23-1	Positive (4.06 Log IU/mL)	Positive (4.32 Log IU/mL)
HCV23-2	Positive (3.69 Log IU/mL)	Positive (3.97 Log IU/mL)
HCV23-3	Negative	Negative

The qualitative results are the same and the quantitative results are similar (<10% difference), therefore the samples were considered as stable.

## 1.2 Scoring system

The scoring system is based on the scoring system of QCMD.

### 1.2.1 QUALITATIVE RESULTS

Table 1.2. Qualitative scoring system.

Sample type	Result	Score
Negative	Negative	0
Negative	Positive	3
Negative	Not determined	3
Frequently detected	Positive	0
Frequently detected	Negative/Not determined	3
Detected	Positive	0
Detected	Negative/not determined	2
Infrequently detected	Positive	0
Infrequently detected	Negative/not determined	1

Frequently detected if detected by more than 95% of the participants.

Detected if detected by more than 65% of the participants.

Infrequently detected if detected by less or equal to 65% of the participants.

### 1.2.2 QUANTITATIVE RESULTS

A Z-score is calculated

$$Z = R - TV / SD$$

R= result

TV=target value= the median of all the results

SD=standard deviation= (Percentile 75-percentile 25)/1,349

If  $|Z| < 1$ , the score is 0

If  $1 \leq |Z| < 2$ , the score is 1

If  $2 \leq |Z| < 3$  the score is 2

If  $|Z| \geq 3$ , the score is 3 and the laboratory is cited.

## 1.3 Survey dates

Sending of the samples: 26/09/2023

Closure date: 10/10/2023

Preliminary report: 17/10/2023

## 2 RESULTS

### 2.1 HBV

#### 2.1.1 THE PARTICIPANTS

25 laboratories sent results, 24 sent quantitative results and 1 sent qualitative results only.

#### 2.1.2 RESULTS PER SAMPLE

##### 2.1.2.1 Qualitative results

Table II.1. Qualitative results.

Sample ID	Expected qualitative result	Observed qualitative results
HBV23-1	Positive	25 positive results
HBV23-2	Positive	25 positive results
HBV23-3	Negative	25 negative results

All the 25 participants obtained the expect results for the qualitative detection of HBV in the serum.

##### 2.1.2.2 Quantitative results

24 laboratories encoded quantitative results for the 2 positive samples. The median of all the results per sample was calculated and used as target value to calculate Z scores:  $Z=R-T/SD$  where R=result, T: target, SD: standard deviation. A Z score below 3 is considered as acceptable and a Z score upper or egal to 3 is unacceptable and means that the result was incorrect.

Table II.2. quantitative results

Sample ID	Median±SD (Log <sub>10</sub> IU/mL)	Z<1	1≤Z<2	2≤Z<3	Z≥3	Comment
HBV23-1	6.52±0.11	16	6	0	2	2 incorrect results
HBV23-2	5.445±0.078	16	2	0	6	6 incorrect results

Out of the 48 results (24 per sample), 40 (83.3%) were acceptable (Z<3) and 8 (16.7%) were incorrect (Z≥3).

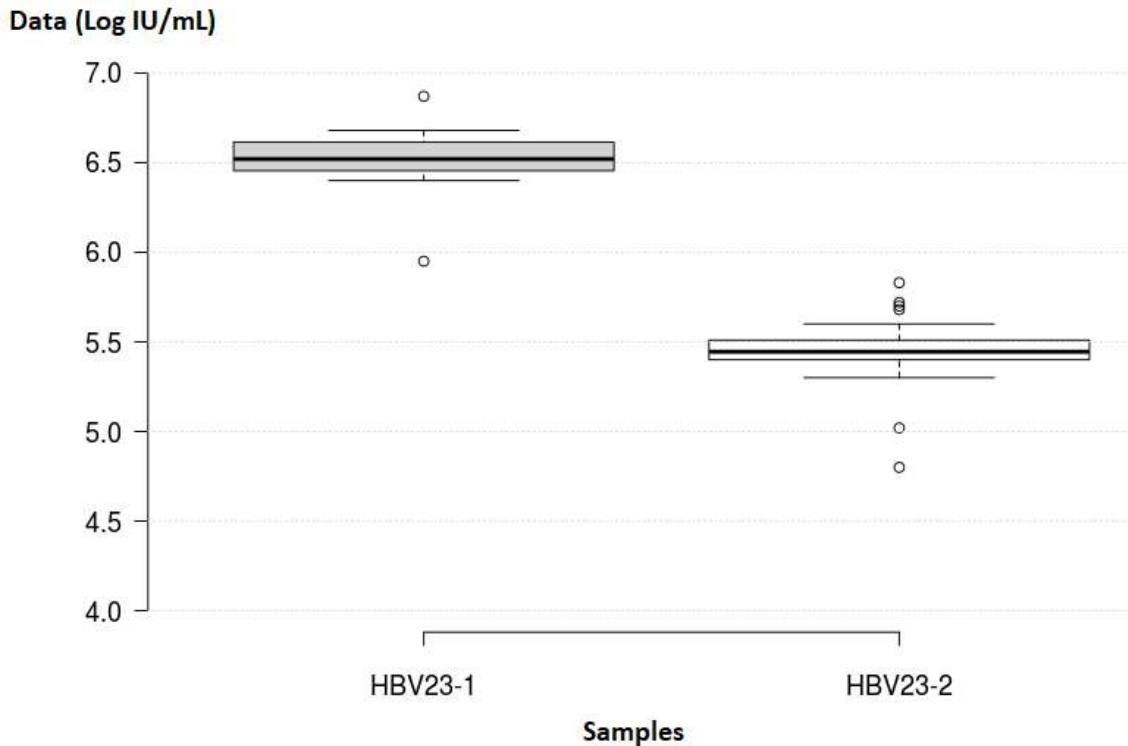


Figure 1. Distribution of results (boxplots) per sample.

### 2.1.3 RESULTS PER METHOD

Table II.3. Quantitative results per method

Method	N	NR	NCR	%	Z<1	1<Z<2	2<Z<3	Z≥3	ranking
Cobas 5800 HBV test	2	4	4	100	4	0	0	0	1
Cobas 6800 HBV	2	4	4	100	4	0	0	0	1
NeuMoDx HBV quant Assay	1	2	2	100	2	0	0	0	1
Cepheid Xpert HBV viral load	9	18	17	94.4	16	1	0	1	2
Abbott ALINITY M HBV AMP KIT	7	14	10	71.4	4	6	0	4	3
Aptima HBV Quant assay	1	2	1	50	1	0	0	1	4
In house	2	4	2	50	1	1	0	2	4
Total	24	48	40	83.3	32	8	0	8	

## 2.2 HCV

### 2.2.1 THE PARTICIPANTS

32 laboratories sent results for HCV; 30 sent quantitative results and 2 sent qualitative results only.

### 2.2.2 QUALITATIVE RESULTS

All the participants find the samples HCV23-1 and HCV23-2 as positive and the sample HCV23-3 as negative.

### 2.2.3 QUANTITATIVE RESULTS

#### 2.2.3.1 Results per sample

Table II.4. Quantitative results per sample.

Sample ID	Median±SD (Log <sub>10</sub> IU/mL)	Z<1	1≤Z<2	2≤Z<3	Z≥3	Comment
HCV23-1	4.045±0.22	20	8	2	0	ok
HCV23-2	3.71±0.13	22	7	1	0	ok

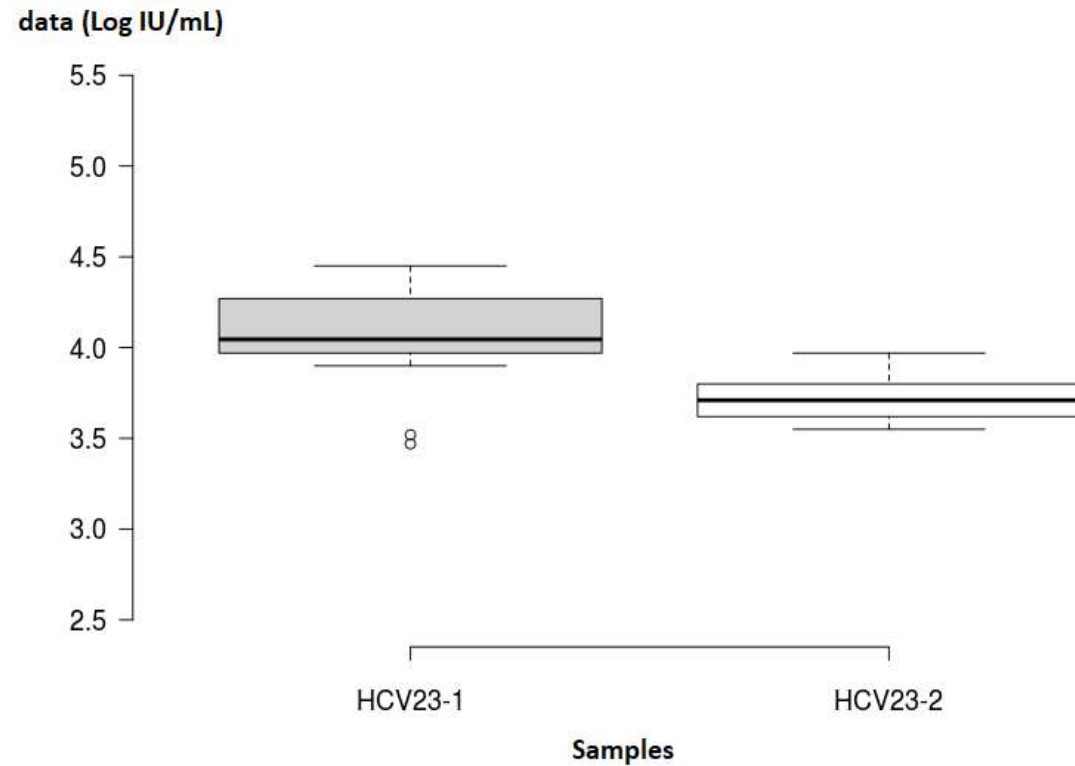


Figure 2. Distribution of the results (boxplots) per sample.



### 2.2.3.2 Results per method

Table II.5. Quantitative results per method.

Method	N	NR	NCR	%	Z<1	1<Z<2	2<Z<3	Z>3
Cobas 4800 HBV test	1	2	2	100	1	1	0	0
Cobas 5800 HCV test	2	4	4	100	2	2	0	0
Cobas 6800 HCV test	4	8	8	100	5	3	0	0
Cepheid Xpert HCV viral load	14	28	28	100	24	4	0	0
Abbott ALINITY M HCV AMP KIT	7	14	14	100	9	4	1	0
Aptima HCV Quant assay	2	4	4	100	1	1	2	0
Total	30	60	60	100	42	15	3	0

### 2.2.4 GENOTYPING

Only 10 laboratories sent results for the genotyping of HCV.

Table II.6. Genotyping results.

Sample	Expected genotype*	Obtained results
HCV23-1	3a	7 answers 3a 3 answers 3
HCV23-1	4f	5 answers 4f 1 answer 4 1 answer 4 and 5 1 answer non-1 1 answer "not determined" 1 answer "Invalid"

- As determined by the NRC

Used methods:

4 laboratories used the HCV genotype LIPA 2.0  
 2 laboratories used a Sanger sequencing method  
 2 used a real-time qPCR method  
 2 used a NGS method (Nanopore or Ion torrent)  
 The individual results are presented in the annexes.

### 3 ANNEXES

#### 1. Individual results for HBV detection and/or quantification.

Table III.1. Individuals results for the HBV detection and quantification.

Sample →	HBV2301		HBV2302		HBV2303
Status →	Positive		Positive		Negative
Target value Median (Log IU/mL) ->	6.52		5.445		NA
Detection/quantification kit		Z1-score		Z2-score	
Abbott ALINITY M HBV AMP KIT	6.63	1.00	5.72	3.53	NEG
Abbott ALINITY M HBV AMP KIT	6.53	0.09	5.42	-0.32	NEG
Abbott ALINITY M HBV AMP KIT	6.55	0.27	5.52	0.96	NEG
Abbott ALINITY M HBV AMP KIT	6.68	1.45	5.7	3.27	NEG
Abbott ALINITY M HBV AMP KIT	6.67	1.36	5.83	4.94	NEG
Abbott ALINITY M HBV AMP KIT	6.65	1.18	5.68	3.01	NEG
Abbott ALINITY M HBV AMP KIT	6.4	-1.09	5.3	-1.86	NEG
Aptima HBV Quant Assay	6.49	-0.27	5.02	-5.45	NEG
Cepheid Xpert HBV viral load	6.56	0.36	5.43	-0.19	NEG
Cepheid Xpert HBV viral load	6.59	0.64	5.5	0.71	NEG
Cepheid Xpert HBV viral load	6.42	-0.91	5.43	-0.19	NEG
Cepheid Xpert HBV viral load	6.5	-0.18	5.45	0.06	NEG
Cepheid Xpert HBV viral load	6.51	-0.09	5.48	0.45	NEG
Cepheid Xpert HBV viral load	6.49	-0.27	5.5	0.71	NEG
Cepheid Xpert HBV viral load	6.53	0.09	5.43	-0.19	NEG
Cepheid Xpert HBV viral load	6.46	-0.55	5.38	-0.83	NEG
Cepheid Xpert HBV viral load	6.87	3.18	5.6	1.99	NEG
Cobas 5800 HBV test	6.42	-0.91	5.37	-0.96	NEG
Cobas 5800 HBV test	6.44	-0.73	5.44	-0.06	NEG
Cobas 6800 HBV	6.45	-0.64	5.4	-0.58	NEG
Cobas 6800 HBV	6.5	-0.18	5.46	0.19	NEG
In house	6.64	1.09	5.5	0.71	NEG
In house	5.95	-0.18	4.8	-0.27	NEG
NeuMoDx HBV quant Assay	6.6	0.73	5.4	-0.58	NEG
MULTIPLEX HIV HCV HBV ROCHE 6800*	POS		POS		NEG

\*: qualitative detection only

POS: positive

NEG: Negative

NA : not applicable

Color code : green :  $|Z| < 1$  ; yellow :  $1 \leq |Z| < 2$  ; red :  $|Z| \geq 3$

2. Individual results for HCV detection and/or quantification.

Table III.2. Individual results for HCV detection and/or quantification.

Samples →	HCV2301		HCV2302		HCV2303
Status →	Positive		Positive		Negative
Target value (Log IU/mL) →	4.045		3.71		NA
Detection/quantification Method		Z1		Z2	
Abbott Allinity m HCV amp kit	4.26	0.98	3.71	0.00	NEG
Abbott Alinity m HCV amp kit	4.36	1.43	3.82	0.85	NEG
Abbott Alinity m HCV amp kit	4.04	-0.02	3.69	-0.15	NEG
Abbott Alinity m HCV amp kit	4.24	0.89	3.9	1.46	NEG
Abbott Alinity m HCV amp kit	4.32	1.25	3.97	2.00	NEG
Abbott Alinity m HCV amp kit	4.17	0.57	3.69	-0.15	NEG
Abbott Alinity m HCV amp kit	4	-0.20	3.57	-1.08	NEG
APTIMA HCV quant dx assay	3.47	-2.61	3.55	-1.23	NEG
Cepheid Xpert HCV viral load	3.97	-0.34	3.77	0.46	NEG
Cepheid Xpert HCV viral load	3.96	-0.39	3.55	-1.23	NEG
Cepheid Xpert HCV viral load	4	-0.20	3.6	-0.85	NEG
Cepheid Xpert HCV viral load	3.97	-0.34	3.69	-0.15	NEG
Cepheid Xpert HCV viral load	4	-0.20	3.82	0.85	NEG
Cepheid Xpert HCV viral load	3.97	-0.34	3.8	0.69	NEG
Cepheid Xpert HCV viral load	4.08	0.16	3.7	-0.08	NEG
Cepheid Xpert HCV viral load	3.9	-0.66	3.56	-1.15	NEG
Cepheid Xpert HCV viral load	3.91	-0.61	3.71	0.00	NEG
Cepheid Xpert HCV viral load	4.05	0.02	3.82	0.85	NEG
Cepheid Xpert HCV viral load	3.95	-0.43	3.59	-0.92	NEG
Cepheid Xpert HCV viral load	4.14	0.43	3.88	1.31	NEG
Cepheid Xpert HCV viral load	3.92	-0.57	3.88	1.31	NEG
Cepheid Xpert HCV viral load	4.01	-0.16	3.71	0.00	NEG
Aptima HCV quant Dx	3.52	-2.39	3.72	0.08	NEG
Roche Cobas 4800 HCV	4.45	1.84	3.61	-0.77	NEG
Roche Cobas 5800 HCV	4.33	1.30	3.62	-0.69	NEG
Roche Cobas 5800 HCV	4.32	1.25	3.74	0.23	NEG
Roche Cobas 6800 HCV	4.35	1.39	3.72	0.08	NEG
Roche Cobas 6800 HCV	4.16	0.52	3.62	-0.69	NEG
Roche Cobas 6800 HCV	4.35	1.39	3.72	0.08	NEG
Roche Cobas 6800 HCV	4.27	1.02	3.64	-0.54	NEG
Procleix Ultrio Elite Assay HCV*	POS		POS		NEG
Roche Cobas 6800 multiplex *	POS		POS		NEG

\*: qualitative detection only

POS: positive

NEG: Negative

NA : not applicable

Color code : green :  $|Z| < 1$  ; yellow :  $1 \leq |Z| < 2$  ; orange:  $2 \leq |Z| < 3$

### 3. HCV genotyping.

Table III.3. Individual genotyping results.

Samples →	HCV2301	HCV2302
Genotyping method	genotype	genotype
Abbott m2000RT HCV genotype II	3	4 et 5
HCV genotype LIPA	3a	Non 1
in house sequencing (ABI3500)	3a	ND
NGS : nanopore sequencing	3a	4f
NGS: Sentosa SQ301 (ion torrent)	3a	4f
RT-qPCR cobas 4800	3	4
Sanger sequencing	3	invalid
Versant HCV genotype LIPA	3a	4F
Versant HCV genotype LIPA	3a	4f
Versant HCV genotype LIPA	3a	4f

**END**

© Sciensano, Brussels 2023.

This report may not be reproduced, published or distributed without the consent of Sciensano. The laboratories individual results are confidential. They are not passed on by Sciensano to third parties, nor to members of the Commission, the committees of experts or the working group EQA.